


# How to use the baby scale

DS-B03


★ Please measure weight first and then height.

Before turn the power on, position the touch-bar at the right end (start point).


**1** To turn the power on push the button for 2 sec. and release it when "----" shown.

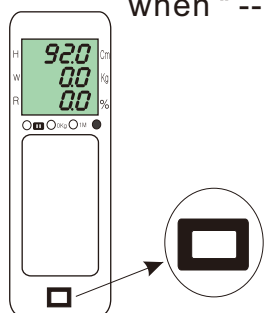


**2** Place a baby in the weighing board.  
 ※ Please do not leave the baby alone.

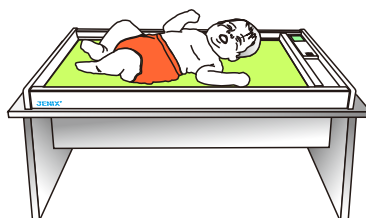


**3** Wait a moment (or push the button) until the weight value is fixed.

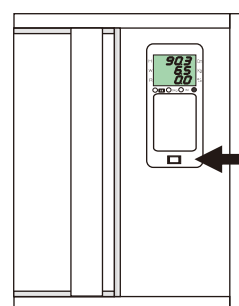

**4** To measure height, have the sole of the baby's feet contacted the left wall and straighten the both knees.

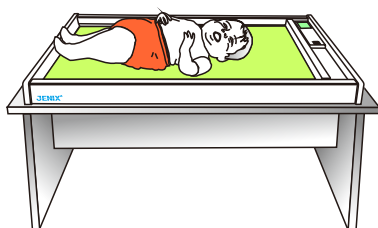

**5** Move the touch-bar until it touch the top of the baby's head.

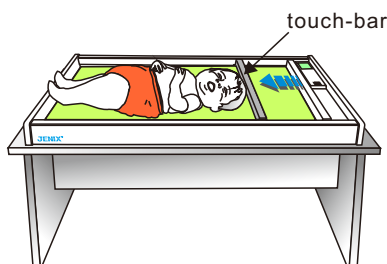

**6** Measuring is now finished. Please check the result.

H 65.2 cm → height  
 W 6.3 kg → weight  
 R 16.5 % → BMI (KAUP index)

BMI criterion	
~13	: Starvation
13~15	: Underweight
15~18	: Normal
18~20	: Overweight
20~	: Obese



Please be careful

1. Don't leave a baby in the scale to avoid falling accident.
2. When you put a thing under 1.0kg in the weighing board, soon it automatically turns to 0.0kg. This function is useful when you use a blanket or basket.
3. Don't push the button strongly and for more than 3 sec when starting. (weight error)
4. Please pay attention not to touch any part of the scale while measuring.